RAPIDIAG CONTROL+ VRS

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DEFINITION

RAPIDIAG CONTROL + has been designed for the use with immunochromatographic **RAPIDIAG VRS** test.

This control + is designed to produce a positive reaction with **RAPIDIAG VRS** strips.

This control allows to verify the presence and the specificity of antibodies directed against the VRS antigens.

RAPIDIAG CONTROL		Antibody
COR C-1086	RAPIDIAG CONTROL + VRS	Anti-VRS (COR C-1006)

PRINCIPLE

When the strip is dipped in the control diluted with the extraction buffer of the kit, the solution migrates by passive diffusion and the control reagent binds first to a conjugated antibody then to the antibody coated on the test line, thereby causing a coloured line to appear on the specific test line and allows to valid the test.

REAGENTS

RAPIDIAG CONTROL +	CONTROL +	1mL
--------------------	-----------	-----

PRECAUTIONS

This reagents consists of non-hazardous virus lysate.

STABILITY AND STORAGE

- Store the reagents at +2°C to +8°C when not in use.
- Do not use reagents beyond the expiration date.

PROTOCOL FOR USE

RAPIDIAG CONTROL + reagent should be diluted ½ with the extraction buffer of the kit.

RAPIDIAG CONTROL + reagent should be analyzed in the same manner as patient samples, according to the instructions supplied.

RESULTS INTERPRETATION

The results are to be interpreted as follows

1 upper line = negative

2 lines = positive

0 line = invalid*

* *The absence of the control line, which is the upper line, makes the result invalid. In this case, the sample must be retested.*

SYMBOLS USED

	Temperature limitation
	In Vitro Diagnostic Use
	EC Declaration of Conformity
	Catalog Number
	Lot Number
	Use by

biomédical diagnostics

Office
Actipole 25
4 bld de Beaubourg
77435 Marne la Vallée Cx2
France

Tel : 33 1 64 62 10 12
Fax : 33 1 64 62 09 66

E-mail : support@bmd-net.com
Internet : www.bmd-net.com