# ELISIS ENA Jo-1 IgG

REF LIS 1113

## DÉFINITION

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## BIBLIOGRAPHIE

---

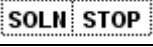
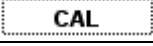
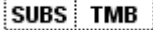
- NISHIKAI AND REICHLIN (1980).  
Heterogeneity of precipitating antibodies in polymyositis and dermatomyositis. Characterization of the Jo-1 antibody system.  
Arthritis Rheum 23: 881-888.
- LOVE LA, LEFF RL, FRASER DD, TARGOFF IN, DALAKAS M, PLOTZ PH, MILLER FW (1991).  
A new approach to the classification of idiopathic inflammatory myopathy: myositis-specific autoantibodies define useful homogeneous patient groups.  
Medicine (Baltimore) 70: 360-374.
- MILLER FW, TWITTY SA, BISWAS T, PLOTZ PH (1990a).  
Origin and regulation of a disease-specific autoantibody response. Antigenic epitopes, spectro type stability and isotype restriction of anti-Jo-1 autoantibodies.  
J Clin Invest 85: 468-475.
- MILLER FW (1991).  
Humoral immunity and immunogenetics in the idiopathic inflammatory myopathies.  
Curr Opin Rheumatol 3: 902-019.
- BISWAS T, MILLER FW, TAKAGAKI Y, PLOTZ PH (1987).  
An enzyme-linked immunosorbent assay for the detection and quantitation of anti-Jo-1 antibody in human serum.  
J Immunol Methods 98: 243-248.

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REF LIS 1113

## DEFINITION

**ELISIS ENA Jo-1 IgG kit** () is a solid phase enzyme immunoassay with recombinant human histidyl-tRNA-synthetase (HRS) for the quantitative and qualitative detection of antibodies against Jo-1 (named after the prototype patient) in human serum.

The results of the **ELISIS ENA Jo-1 IgG kit** () are to be used in conjunction with the clinical findings and the other laboratory tests to aid in the diagnosis of polymyositis and dermatomyositis.

**ELISIS ENA Jo-1 IgG kit** () may be used with the **CARIS™ system** (diluting and dispensing device).

## DIAGNOSTIC VALUE

Antibodies against Jo-1 are directed against the reactive site of histidyl-tRNA-synthetase (HRS) which is an cytoplasmic enzyme belonging to the group of aminoacyl transferases. These are responsible for the linking of the respective amino acid (for HRS it is histidine) to its cognate transfer RNA. HRS is present in the cell as homodimer, its identical subunits of approximately 50 kDa are each bound to tRNA.

Autoantibodies are commonly found in sera with myositis, and some are highly specific for this disorder. Each myositis-specific antibody defines a group of myositis patients with distinctive clinical features. About 30 % of adults with myositis have antibodies to an aminoacyl transferase, and in at least 80% of cases the antibodies are directed to HRS. Anti-Jo-1 antibodies are almost exclusively found in patients with myositis. They occur in primary polymyositis with a prevalence of 33%, in primary dermatomyositis with 25% and in secondary myositis associated with other connective tissue diseases with 15% prevalence. The onset of the disease is often acute with prominent systemic features such as fever. Myositis is often severe although cases without clinical muscle involvement are reported. Interstitial pneumonitis is a prominent clinical manifestation which is the next most common clinical feature after myositis in anti-Jo-1 positive patients (present in 50-90%).

Other myositis-specific antibodies have been detected (prevalence < 5%): antibodies against threonyl-(anti-PL-7), alanyl- (anti-PL-12), isoleucyl- (anti-OJ) and glycyl-tRNA synthetase (anti-EJ) e.g.

## ASSAY PRINCIPLE

Recombinant human histidyl-tRNA-synthetase (Jo-1) is coated onto a polystyrene microtiter plate (12 strips of 8 wells).

- First, the diluted sample is added to the coated well, which allows to bind. After incubation, unbound proteins are removed by washing.
- Then, a monoclonal anti-human IgG conjugated to horseradish peroxidase is added to each well. The conjugate binds to the antigen-antibody complex. After incubation, the excess conjugate is removed by a second wash.
- The chromogenic step is performed by the addition of TMB substrate (3,3',5,5' – tetramethylbenzidine). During this step, a colour will develop in proportion to the amount of antibodies in the sample.
- Addition of Stop Solution HCl (1 M) serves to stop the enzymatic reaction.
- After stopping the reaction by HCl (1 M) the optical density is read by a spectrophotometer at 450nm
- The rate of color formation from the chromogen is a function of the amount of conjugate bound to the antigen-antibody complex and this is proportional to the initial concentration of the respective antibodies in the patient sample.

## SAMPLES COLLECTION AND HANDLING

- The test should be performed on serum. Use preferentially freshly collected serum samples. Blood withdrawal must follow national requirements.
- Lipemic sera should be avoided, as well as samples which have been frozen and defrosted more than once.
- If determination is not performed immediately, samples should be stored at +2°C/+8°C for no longer than a week or frozen.
- To avoid any non-specific fixation, samples which have been frozen for more than 6 months or which are cloudy, should be centrifuged and filtered.

## STABILITY AND STORAGE

- Store reagents and assay strips at +2°C/+8°C in their own package.
- Do not use kits beyond the expiration date.
- Store unused strips into their plastic bag with the desiccant.
- Store all components immediately after use again at +2°C/+8°C.

## REAGENTS

96 wells microplate coated with recombinant human histidyl-tRNA-syn-thetase (Jo-1) (12x8 individual breakaway microwells) <span style="float: right;">MP</span>	12 strips
1 vial of Cut-off Calibrator (capped blue: yellow solution) containing: human serum (diluted), sodium azide < 0.1% (preservative) <u>Ready to use</u> <span style="float: right;">CONTROL REF</span>	1 x 1.5mL
6 vials of Calibrators (color increasing with concentration: yellow solutions) 0, 3, 10, 30, 100, 300 U/mL containing: human serum (diluted), sodium azide < 0.1% (preservative) <u>Ready to use</u> <span style="float: right;">CAL</span>	6 x 1.5mL
1 vial of Positive control (capped red: yellow solution) containing: human serum (diluted), sodium azide < 0.1% (preservative) <u>Ready to use</u> <span style="float: right;">CONTROL +</span>	1 x 1.5mL
1 vial of Negative control (capped green: yellow solution) containing: human serum (diluted), sodium azide < 0.1% (preservative) <u>Ready to use</u> <span style="float: right;">CONTROL -</span>	1 x 1.5mL
1 vial of anti-human IgG-HRP Conjugate (capped blue: blue solution) <u>Ready to use</u> <span style="float: right;">CONJ IgG</span>	1 x 15mL
1 vial of TMB Substrate (capped black) containing: stabilized TMB/ H <sub>2</sub> O <sub>2</sub> <u>Ready to use</u> <span style="float: right;">SUBS TMB</span>	1 x 15mL
1 vial of Sample Diluent - 5x concentrated (capped white: yellow solution) containing: Tris, NaCl, BSA, sodium azide < 0.1% (preservative) <u>To dilute</u> <span style="float: right;">DIL SPE 5x</span>	1 x 20mL
1 vial of Wash Buffer - 50x concentrated (capped white: green solution) containing: Tris, NaCl, Tween 20, sodium azide < 0.1% (preservative) <u>To dilute</u> <span style="float: right;">BUF WASH 50x</span>	1 x 20mL
1 vial of Stop Solution (capped white: colorless solution) containing: 1M Hydrochloric Acid <u>Ready to use</u> <span style="float: right;">SOLN STOP</span>	1 x 15mL

**ELISIS ENA Jo-1 IgG kit** ([bmd](#)) is calibrated against reference sera from the CDC (Centres for Disease Control and Prevention) Atlanta. The results are expressed in AU/mL.

## MATERIAL REQUIRED BUT NOT PROVIDED

- Purified water (according to the definition of the United States Pharmacopeia and the European Pharmacopeia)
- Microtiter plate reader 450nm reading filter and optional 620nm reference filter (600-690nm)
- Glass ware (cylinder 100-1000mL)
- Test tubes for serum dilutions
- Vortex mixer
- Precision pipettes (10, 100, 200, 500, 1000µL) or adjustable multipipette (100-1000µL)
- Microplate washing device (300µL repeating or multichannel pipette or automated System)
- Adsorbent paper

## SETUP

All the reagents should be prepared as required:

### 1. Preparation of Sample Diluent Solution

- Dilute concentrated Sample Diluent 1/5 in distilled water.  
Ex: 20mL plus 80mL
- Storage period: 1 month at +2°C/+8°C.

### 2. Preparation of wash buffer

- Dilute concentrated Wash buffer 1/50 in distilled water.  
Ex: 20mL plus 980mL
- Prepare 20mL of diluted wash buffer for 8 wells or 200mL for 96 wells.
- Storage period: 1 month at +2°C/+8°C.

### 3. Preparation of samples

- Dilute to 1/101 in Sample Diluent (1x)  
Ex: 1000µL Sample Diluent (1x) + 10µL serum
- Shake vigorously at the vortex.

### 4. Washing Step

- ❖ Automated washing:
  - Consider excess volumes required for setting up the instrument and dead volume of robot pipette.
- ❖ Manual washing:
  - Discard liquid from wells by inverting the plate.
  - Knock the microwell frame with wells downside vigorously on clean adsorbent paper.
  - Pipette 300µl of diluted wash buffer into each well, wait for 20 seconds.
  - Repeat the whole procedure twice again.

### 5. Microplates

- Calculate the number of wells required for the test.
- Remove unused wells from the frame, replace and store in the provided plastic bag, together with desiccant, seal tightly (+2°C/+8°C).

## PRECAUTIONS

Allow all reagents and samples to come to room temperature (+18°C / +25°C) before handling.

Check that all plates are well drained after each wash.

Do not smoke, eat or drink when manipulating the kit.

Do not pipette by mouth.

*Human sources for the preparation of calibrators and controls have been tested and found negative for antibody to HIV 1 and 2, antibody to hepatitis C virus and hepatitis B virus antigen. Nevertheless, no test can offer complete assurance that HIV, hepatitis B virus or other infectious agents are absent. Therefore, the reagents should be handled as potentially infective materials.*

This kit contains potentially hazardous components. Though kit reagents are not classified being irritant to eyes and skin we recommend to avoid contact with eyes and skin and wear disposable gloves.

Calibrators, Controls and Buffers contain less of 0.1% (w/v) sodium azide. Do not eat and avoid contact with skin and eyes. This substance can form explosive mixtures in copper or lead piping. Rinse thoroughly after flushing.

**ELISIS ENA Jo-1 IgG kit** ([bmd](#)) has been developed according CE Directives 67/548/EEC and 1999/45/EC relating to the classification, packaging and labeling of dangerous preparations.

**ELISIS ENA Jo-1 IgG kit** ([bmd](#)) has been optimized for the use as describe in this procedure. Do not substitute other manufacturer's reagents. Dilution or adulteration of these reagents may also affect the performance of the test. Close adherence to the test procedure will assure optimal performance.

Do not mix or substitute reagents or microplates from different lot numbers. This may lead to variations in the results.

**Incubation: We recommend test performance at +30°C for automated Systems.**

Never expose components to higher temperature than +37°C.

Always pipette Substrate and Conjugate with brand new tips only.

Protect Substrate from light.

## METHOD

### 1. Preparing the test

Use the work sheet to note the sample locations.

Detach the exact number of wells needed and return unused wells to plastic pouch provided in the kit, with the desiccant bag.

**For quantitative interpretation use calibrators to establish a standard curve**

Set out in duplicate:

- Calibrators (Cal A, Cal B, Cal C, Cal D, Cal E and Cal F)
- Positive control (Ctrl +)
- Negative control (Ctrl -)
- Diluted samples (S)

	1	2	3	4	5	6	7	8	9	10	11	12
A	Cal A	Cal E	S1	S9								
B	Cal A	Cal E	S2	S10								
C	Cal B	Cal F	S3	S11								
D	Cal B	Cal F	S4	S12								
E	Cal C	Ctrl +	S5	S13								
F	Cal C	Ctrl +	S6	S14								
G	Cal D	Ctrl -	S7									
H	Cal D	Ctrl -	S8									

**For qualitative interpretation use cut-off calibrators**

Set out in duplicate:

- Negative control (Ctrl -)
- Cut-off calibrator (Ctrl Ref)
- Positive control (Ctrl +)
- Diluted samples (S)

	1	2	3	4	5	6	7	8	9	10	11	12
A	Ctrl -	S3	S11	S19								
B	Ctrl -	S4	S12	S20								
C	Ctrl Ref	S5	S13	S21								
D	Ctrl Ref	S6	S14	S22								
E	Ctrl +	S7	S15	S23								
F	Ctrl +	S8	S16	S24								
G	S1	S9	S17									
H	S2	S10	S18									

### 2. Sample incubation

Add 100µL of Calibrators OR Cut-off Calibrator and controls into the designated microwells

Add 100µL of diluted samples

Incubate for 30 minutes at room temperature (+20°C/+32°C)

*Wash step:*

Remove the contents of the wells by rapid inversion

Wash 3 times with 300µL of washing buffer

### 3. Incubation of conjugate

Add 100µL of conjugate

Incubate for 30 minutes at room temperature (+20°C/+32°C)

*Wash step:*

Remove the contents of the wells by rapid inversion

Wash 3 times with 300µL of washing buffer

### 4. Incubation of substrate

Add 100µL TMB substrate into each well

Incubate for 30 minutes at room temperature (+20°C/+32°C)

Protect the wells from intense light

### 5. Stop solution

Add 100µL of Stop solution to each well

Incubate for 5 minutes minimum

Agitate plate carefully for 5 sec.

### 6. Reading

Read the optical density of each well at 450nm (optionally 450/620nm) within 30 minutes.

## INTERPRETATION OF RESULTS

For **quantitative interpretation** establish the standard curve by plotting the **optical density (OD) of each calibrator (y-axis)** with respect to the corresponding concentration values in **AU/mL (x-axis)**. For best results [bmd](#) recommend log (x) / lin (y) coordinates and 4-Parameter Fit. From the OD of each sample, read the corresponding antibody concentrations expressed in **AU/mL**.

Negative Results	Positive Results
≤ 15 AU/mL	> 15 AU/mL

### Example of a standard curve

We recommend pipetting calibrators in parallel for each run.

Calibrators IgG	OD 450/620nm	CV % (Variation)
0 AU/ml	0.051	0.0
3 AU/ml	0.136	1.8
10 AU/ml	0.334	2.2
30 AU/ml	0.635	2.9
100 AU/ml	1.278	2.4
300 AU/ml	2.292	0.8

### Example of calculation

Patient	Replicate (OD)	Mean (OD)	Result (AU/mL)
P01	0.840/0.849	0.845	48.4
P02	0.351/0.376	0.364	13.6

### Do not use this example for interpreting patients results!

For lot specific data, see enclosed quality control certificate.

Each laboratory should establish its own normal range based upon its own techniques, controls, equipment and patient population according to their own established procedures.

For **qualitative interpretation** read the optical density of the cut-off calibrator and the patient samples. Compare patient's OD with the OD of the cut-off calibrator. All samples which are higher than cut-off are considered positive.

<b>Negative:</b>	OD Patient < OD cut-off
<b>Positive:</b>	OD Patient > OD cut-off

**A definite clinical diagnosis should not be based on the results of the performed test only, but should be made by the physician after all clinical and laboratory findings have been evaluated. The diagnosis is to be verified using different diagnostic methods.**

## LIMITATION

Hemolytic, lipemic, icteric samples or contaminated samples with may confound the results of this assay.

## QUALITY CONTROL

The kit contains positive and negative controls. Both positive and negative control should be included with each run of the test. Positive and negative controls concentration should fall within the range printed on the quality control certificate. If either reagent control is invalid, the test should be repeated.

[bmd](#) offers an external multiparameter quality control (IMMUNO-TROL V, Cat: HM052). This control should be handled as samples.

## CHARACTERISTICS AND PERFORMANCES OF THE TEST

### Sensitivity, specific affinity and agreement to other methods

**ELISIS ENA Jo-1 IgG** kit ([bmd](#)) has been compared to one predicate ELISA kit.

The study was performed on 340 samples from patients with associated systemic auto-immune diseases.

### Results

Antigenic specificity	Relative sensitivity (%)	Relative specificity (%)	Concordance (%)
Jo-1	100	100	100

### Test Precision

Estimated for each parameter on three positive samples : strongly, moderate and weakly positive.

	ELISIS ENA Jo-1 IgG			
	Intra-assay (24 tests in a same assay)		Inter-assay (18 tests in 3 differents assays)	
	Titer mean AU/mL	CV (%)	Titer mean AU/mL	CV (%)
Jo-1	112.9	9.3	112.9	9.3
	63.3	7.5	63.3	7.5
	15.3	1.0	15.3	1.0

### Threshold value

Calculated on 40 samples from « blood donors ».

ELISIS ENA Jo-1 IgG	
Cut-off : Mean value +3SD	15 AU/mL

## REFERENCES

---

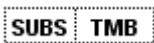
- NISHIKAI AND REICHLIN (1980).  
Heterogeneity of precipitating antibodies in polymyositis and dermatomyositis. Characterization of the Jo-1 antibody system.  
*Arthritis Rheum* 23: 881-888.
- LOVE LA, LEFF RL, FRASER DD, TARGOFF IN, DALAKAS M, PLOTZ PH, MILLER FW (1991).  
A new approach to the classification of idiopathic inflammatory myopathy: myositis-specific autoantibodies define useful homogeneous patient groups.  
*Medicine (Baltimore)* 70: 360-374.
- MILLER FW, TWITTY SA, BISWAS T, PLOTZ PH (1990a).  
Origin and regulation of a disease-specific autoantibody response. Antigenic epitopes, spectro type stability and isotype restriction of anti-Jo-1 autoantibodies.  
*J Clin Invest* 85: 468-475.
- MILLER FW (1991).  
Humoral immunity and immunogenetics in the idiopathic inflammatory myopathies.  
*Curr Opin Rheumatol* 3: 902-019.
- BISWAS T, MILLER FW, TAKAGAKI Y, PLOTZ PH (1987).  
An enzyme-linked immunosorbent assay for the detection and quantitation of anti-Jo-1 antibody in human serum.  
*J Immunol Methods* 98: 243-248.

## SUMMARY OF METHOD

	AMOUNT TO BE DISTRIBUTED	REAGENTS		INCUBATION CONDITIONS
		QUANTITATIVE METHOD	QUALITATIVE METHOD	
INCUBATION OF SAMPLES	100µL	Calibrators A, B, C, D, E and F (in duplicate)	Negative Control (in duplicate)	30 min at RT
	100µL	Positive Control (in duplicate)	Cut-off calibrator (in duplicate)	
	100µL	Negative Control (in duplicate)	Positive Control (in duplicate)	
	100µL	Diluted samples (in duplicate)	Diluted samples (in duplicate)	
WASHING	Wash 3 times with washing buffer (300µL/ well)			
INCUBATION OF CONJUGATE	100µL	Anti-human IgG-HRP conjugate		30 min at RT
WASHING	Wash 3 times with washing buffer (300µL/ well)			
ENZYME REACTION	100µL	TMB Substrate		30 min at RT protected from intense light
STOP REACTION	100µL	Stop solution (1M HCl)		5 min

RT: room temperature (+20°C/+32°C)

## SYMBOLS USED

	Biological risk		Temperature limitation		Catalog Number
	Read instructions for use		In Vitro Diagnostic Use		Lot Number
	Number of tests		Use by		EC Declaration of Conformity
	Sample Diluent		Stop Solution		Cut-off calibrator
	Wash Buffer		Calibrator		Microplate
	Control		Substrate		Conjugate

***BioMédical Diagnostics SA***

**Office**  
Actipole 25  
4 bld de Beaubourg  
77435 Marne la Vallée Cx2  
France

Tel : 33 1 64 62 10 12  
Fax : 33 1 64 62 09 66

E-mail : [support@bmd-net.com](mailto:support@bmd-net.com)  
Internet : [www.bmd-net.com](http://www.bmd-net.com)