ELISIS CARDIOLIPIN IgG / IgM

REF LIS 1204

DÉFINITION

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Editorial. Lupus 1992, 1 : 51-53

FONT J.

Anti-cardiolipin antibodies in patients with autoimmune diseases: isotype distribution and clinical associations.

Clin. Rheum. 1989, 8 : 475-785

HESS D.C.

Anti-cardiolipin antibodies: a study of frequency in TIA and stroke.

Neurology 1991, 23 (suppl) : 129-132

INFANTE RIVARD C. et al.

Lupus anticoagulant, anti-cardiolipin antibodies and fetal loss. A case control study.

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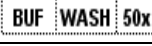
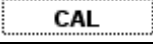
English

DEFINITION

ELISIS CARDIOLIPIN IgG/IgM kit ([bmd](#)) is an enzyme linked immunoassay (ELISA) for the qualitative and quantitative determination of IgG and/or IgM anti-cardiolipin antibodies in human serum samples. Anti-cardiolipin antibodies mainly recognize specific epitopes on a complex composed of cardiolipin and β 2-glycoprotein I which are only expressed when β 2-glycoprotein I interacts with cardiolipin.

The results of the **ELISIS CARDIOLIPIN IgG/IgM kit** ([bmd](#)) are to be used in conjunction with the clinical findings and the other laboratory tests to aid in the diagnosis and risk estimation of thrombosis in patients with systemic lupus erythematosus (SLE).

ELISIS CARDIOLIPIN IgG/IgM kit ([bmd](#)) may be used with the **CARIS™ system** (diluting and dispensing device).

DIAGNOSTIC VALUE

Anti-cardiolipin autoantibodies are present in clinical contexts comparable to those in which lupus anticoagulants (anti-prothrombinase) are found. Moreover, they are frequently associated. In such patients, it is not rare to observe a dissociated syphilitic serology with positive VDRL and negative TPHA. The most frequent associated pathologies are Systemic Lupus Erythematosus, drug-induced lupus, thromboembolic disease, thrombosis in general, recurrent abortion, autoimmune peripheral cytopenia, primary biliary cirrhosis, certain acute infections and HIV and HTLV retrovirus infections. With the exception of infections, in which anti-cardiolipin antibodies seem only transiently present, the antibodies would seem to have a predictive value with respect to thrombosis. They exacerbate the life threat in myocardial infarction and enhance the risk of graft rejection following bypass procedures. The term, "anti-cardiolipin antibody syndrome" has been suggested to designate the morbid signs and symptoms that are associated with the presence of the antibodies. Anti-cardiolipin antibodies are generally of classes IgG, IgG and IgM, or IgG and IgA.

Indications for screening of anti-cardiolipin antibodies

The indications are mainly diagnostic and prognostic. The main clinical indications are:

- ✓ Thrombosis
- ✓ Disseminated Lupus Erythematosus
- ✓ Drug-induced lupus
- ✓ Hematological lupus with anti-mitochondrial antibodies, type 5
- ✓ Soulier-Boffa syndrome
- ✓ Recurrent abortion

At biological level

- ✓ A lupus anticoagulant (anti-prothrombinase) confirmed or expected in the context of an increased activated partial thromboplastin time (APTT)
- ✓ A dissociated syphilitic serology: VDRL positive and TPHA negative
- ✓ Anti-mitochondrial antibodies of types 1 or 5
- ✓ Peripheral cytopenia, particularly that associated with autoimmune disease.

Anti-cardiolipin antibodies, lupus anticoagulant, dissociated syphilitic serology, type 5 anti-mitochondrial antibodies and a positive Coombs test result may be associated in the same patient, but that is rather rare. In contrast, cases in which only one or two antibodies are present are relatively frequent. For that reason, screening for antibodies must be conducted individually, since the absence of one does not necessarily exclude the presence of the other.

Anti-cardiolipin antibodies may be present in normal subjects. If that is the case, they are present at low levels and, frequently, transiently. Their significance is unknown.

ASSAY PRINCIPLE

Highly purified cardiolipin bound to the native human β 2-glycoprotein I is coated onto a polystyrene microtiter plate (12 strips of 8 wells).

- First, the diluted sample is added to the coated well, which allows to bind. After incubation, unbound proteins are removed by washing.
- Then, a monoclonal anti-human IgG or IgM conjugated to horseradish peroxidase is added to each well. The conjugate binds to the antigen-antibody complex. After incubation, the excess conjugate is removed by a second wash.
- The chromogenic step is performed by the addition of TMB substrate (3,3',5,5' – tetramethylbenzidine). During this step, a colour will develop in proportion to the amount of antibodies in the sample.
- Addition of Stop Solution HCl (1 M) serves to stop the enzymatic reaction.
- After stopping the reaction by HCl (1 M) the optical density is read by a spectrophotometer at 450nm
- The rate of color formation from the chromogen is a function of the amount of conjugate bound to the antigen-antibody complex and this is proportional to the initial concentration of the respective antibodies in the patient sample.

SAMPLES COLLECTION AND HANDLING

- The test should be performed on serum. Use preferentially freshly collected serum samples. Blood withdrawal must follow national requirements.
- Lipemic sera should be avoided, as well as samples which have been frozen and defrosted more than once.
- If determination is not performed immediately, samples should be stored at +2°C/+8°C for no longer than a week or frozen.
- To avoid any non-specific fixation, samples which have been frozen for more than 6 months or which are cloudy, should be centrifuged and filtered.

REAGENTS

96 wells microplate coated with highly purified cardiolipin (bovin origin) bound to native human β 2-glycoprotein I (12x8 individual breakaway microwells) MP	12 strips
1 vial of Cut-off Calibrator (capped blue: yellow solution) containing: human serum (diluted), sodium azide < 0.1% (preservative) Ready to use CONTROL REF	1 x 1.5mL
6 vials of Calibrators (color increasing with concentration: yellow solution) 0, 3, 10, 30, 100, 300 GPL/mL or MPL/mL containing: human serum (diluted), sodium azide < 0.1% (preservative) Ready to use CAL	6 x 1.5mL
1 vial of Positive control (capped red: yellow solution) containing: human serum (diluted), sodium azide < 0.1% (preservative) Ready to use CONTROL +	1 x 1.5mL
1 vial of Negative control (capped green: yellow solution) containing: human serum (diluted), sodium azide < 0.1% (preservative) Ready to use CONTROL -	1 x 1.5mL
1 vial of anti-human IgG-HRP Conjugate (capped blue: blue solution) Ready to use CONJ IgG	1 x 15mL
1 vial of anti-human IgM-HRP Conjugate (capped green: green solution) Ready to use CONJ IgM	1 x 15mL
1 vial of TMB Substrate (capped black) Containing: stabilized TMB/H ₂ O ₂ Ready to use SUBS TMB	1 x 15mL
1 vial of Sample Diluent - 5x concentrated (capped white: yellow solution) containing: Tris, NaCl, BSA, sodium azide < 0.1% (preservative) To dilute DIL SPE 5x	1 x 20mL
1 vial of Wash Buffer - 50x concentrated (capped white: green solution) Containing: Tris, NaCl, Tween 20, sodium azide < 0.1% (preservative) To dilute BUF WASH 50x	1 x 20mL
1 vial of Stop Solution (capped white: colorless solution) containing: 1M Hydrochloric Acid) Ready to use SOLN STOP	1 x 15mL

ELISIS CARDIOLIPIN IgG/IgM kit (μ md) is calibrated against reference sera from N.E. Harris, Louisville. The results are expressed in GPL/mL for IgG and in MPL/mL for IgM.

In addition, **ELISIS CARDIOLIPIN IgG/IgM kit** (μ md) is standardized using the Sapporo-Standards: HCAL for IgG and EY2C9 for IgM, reference sera from the CDC (Centers for Disease Control and Prevention) Atlanta.

STABILITY AND STORAGE

- Store reagents and assay strips at +2°C/+8°C in their own package.
- Do not use kits beyond the expiration date.
- Store unused strips into their plastic bag with the desiccant.
- Store all components immediately after use again at +2°C/+8°C.

MATERIAL REQUIRED BUT NOT PROVIDED

- Purified water (according to the definition of the United States Pharmacopeia and the European Pharmacopeia)
- Microtiter plate reader 450nm reading filter and optional 620 nm reference filter (600-690nm)
- Glass ware (cylinder 100-1000mL)
- Test tubes for serum dilutions
- Vortex mixer
- Precision pipettes (10, 100, 200, 500, 1000 μ L) or adjustable multipipette (100-1000 μ L)
- Microplate washing device (300 μ L repeating or multichannel pipette or automated System)
- Adsorbent paper

SETUP

All the reagents should be prepared as required:

1. Preparation of Sample Diluent solution

- Dilute concentrated Sample diluent 1/5 in distilled water.
 Ex: 20mL plus 80mL
- Storage period: 1 month at +2°C/+8°C.

2. Preparation of wash buffer

- Dilute concentrated Wash buffer 1/50 in distilled water.
 Ex: 20mL plus 980mL
- Prepare 20mL of diluted wash buffer for 8 wells or 200mL for 96 wells
- Storage period: 1 month at +2°C/+8°C.

3. Preparation of samples

- Dilute to 1/101 in Sample diluent (1x)
 Ex: 1000 μ L sample buffer (1x) + 10 μ L serum
- Shake vigorously at the vortex.

4. Washing Step

- ❖ Automated washing:
- Consider excess volumes required for setting up the instrument and dead volume of robot pipette.

❖ Manual washing:

- Discard liquid from wells by inverting the plate.
- Knock the microwell frame with wells downside vigorously on clean adsorbent paper.
- Pipette 300µL of diluted wash buffer into each well, wait for 20 seconds.
- Repeat the whole procedure twice again.

5. Microplates

- Calculate the number of wells required for the test.
- Remove unused wells from the frame, replace and store in the provided plastic bag, together with desiccant, seal tightly (+2°C/+8°C).

PRECAUTIONS

Allow all reagents and samples to come to room temperature (+18°C / +25°C) before handling.

Check that all plates are well drained after each wash.

Do not smoke, eat or drink when manipulating the kit.

Do not pipette by mouth.

Human sources for the preparation of calibrators and controls have been tested and found negative for antibody to HIV 1 and 2, antibody to hepatitis C virus and hepatitis B virus antigen. Nevertheless, no test can offer complete assurance that HIV, hepatitis B virus or other infectious agents are absent. Therefore, the reagents should be handled as potentially infective materials.

This kit contains potentially hazardous components. Though kit reagents are not classified being irritant to eyes and skin we recommend to avoid contact with eyes and skin and wear disposable gloves.

Calibrators, Controls and Buffers contain less of 0.1% (w/v) sodium azide. Do not eat and avoid contact with skin and eyes. This substance can form explosive mixtures in copper or lead piping. Rinse thoroughly after flushing.

ELISIS CARDIOLIPIN IgG/IgM kit ([bmd](#)) has been developed according CE Directives 67/548/EEC and 1999/45/EC relating to the classification, packaging and labeling of dangerous preparations.

ELISIS CARDIOLIPIN IgG/IgM kit ([bmd](#)) has been optimized for the use as describe in this procedure. Do not substitute other manufacturer's reagents. Dilution or adulteration of these reagents may also affect the performance of the test. Close adherence to the test procedure will assure optimal performance.

Do not mix or substitute reagents or microplates from different lot numbers. This may lead to variations in the results.

Incubation: We recommend test performance at +30°C for automated Systems.

Never expose components to higher temperature than +37°C.

Always pipette Substrate and Conjugate with brand new tips only.

Protect Substrate from light.

METHOD

1. Preparing the test

Use the work sheet to note the sample locations.

Detach the exact number of wells needed and return unused wells to plastic pouch provided in the kit, with the desiccant bag.

For quantitative interpretation use calibrators to establish a standard curve

Set out in duplicate:

- Calibrators (Cal A, Cal B, Cal C, Cal D, Cal E and Cal F)
- Positive control (Ctrl +)
- Negative control (Ctrl -)
- Diluted samples (S)

	1	2	3	4	5	6	7	8	9	10	11	12
A	Cal A	Cal E	S1	S9								
B	Cal A	Cal E	S2	S10								
C	Cal B	Cal F	S3	S11								
D	Cal B	Cal F	S4	S12								
E	Cal C	Ctrl +	S5	S13								
F	Cal C	Ctrl +	S6	S14								
G	Cal D	Ctrl -	S7									
H	Cal D	Ctrl -	S8									

For qualitative interpretation use cut-off calibrators

Set out in duplicate:

- Negative control (Ctrl -)
- Cut-off calibrator (Ctrl Ref)
- Positive control (Ctrl +)
- Diluted samples (S)

	1	2	3	4	5	6	7	8	9	10	11	12
A	Ctrl -	S3	S11	S19								
B	Ctrl -	S4	S12	S20								
C	Ctrl Ref	S5	S13	S21								
D	Ctrl Ref	S6	S14	S22								
E	Ctrl +	S7	S15	S23								
F	Ctrl +	S8	S16	S24								
G	S1	S9	S17									
H	S2	S10	S18									

2. Sample incubation

Add 100µL of calibrators OR cut-off calibrator and controls into the designated microwells

Add 100µL of diluted samples.

Incubate for 30 minutes at room temperature (+20°C/+32°C)

Wash step:

Remove the contents of the wells by rapid inversion.

Wash 3 times with 300µL of washing buffer.

3. Incubation of conjugate

Add 100µL of conjugate

Incubate for 30 minutes at room temperature (+20°C/+32°C)

Wash step:

Remove the contents of the wells by rapid inversion

Wash 3 times with 300µL of washing buffer

4. Incubation of substrate

Add 100µL TMB substrate into each well

Incubate for 30 minutes at room temperature (+20°C/+32°C)

Protect the wells from intense light

5. Stop solution

Add 100µL of Stop solution to each well

Incubate for 5 minutes minimum

Agitate plate carefully for 5 sec.

6. Reading

Read the optical density of each well at 450 nm (optionally 450/620nm) within 30 minutes.

INTERPRETATION OF RESULTS

For **quantitative interpretation** establish the standard curve by plotting the **optical density (OD) of each calibrator (y-axis)** with respect to the corresponding concentration values in **GPL/mL** or **MPL/ mL (x-axis)**. For best results bmd recommend log (x) / lin (y) coordinates and 4-Parameter Fit. From the OD of each sample, read the corresponding antibody concentrations expressed in **GPL/mL** or **MPL/mL**.

	Negative results	Positive Results
IgG	< 15 GPL/mL	> 15 GPL/mL
IgM	< 15 MPL/mL	> 15 MPL/mL

Example of a standard curve

We recommend pipetting calibrators in parallel for each run.

Calibrators IgG	OD 450/620 nm	CV % (Variation)
0 GPL/ml	0.066	3.2
3 GPL/ml	0.162	0.4
10 GPL/ml	0.291	1.7
30 GPL/ml	0.597	1.3
100 GPL/ml	1.101	2.9
300 GPL/ml	2.039	0.4

Example of calculation

Patient	Replicate (OD)	Mean (OD)	Result (GPL/mL)
P01	0.772/0.752	0.762	48.8
P02	1.058/1.038	1.048	82.9

Do not use this example for interpreting patients results!

For lot specific data, see enclosed quality control certificate.

Each laboratory should establish its own normal range based upon its own techniques, controls, equipment and patient population according to their own established procedures.

For **qualitative interpretation** read the optical density of the cut-off calibrator and the patient samples. Compare patient's OD with the OD of the cut-off calibrator. All samples which are higher than cut-off are considered positive.

Negative:	OD Patient < OD cut-off
Positive:	OD Patient > OD cut-off

A definite clinical diagnosis should not be based on the results of the performed test only, but should be made by the physician after all clinical and laboratory findings have been evaluated. The diagnosis is to be verified using different diagnostic methods.

LIMITATION

Hemolytic, lipemic, icteric samples or samples with abnormal concentration may confound the results of this assay.

QUALITY CONTROL

The kit contains positive and negative controls. Both positive and negative control should be included with each run of the test. Positive and negative controls concentration should fall within the range printed on the quality control certificate. If either reagent control is invalid, the test should be repeated.

bmd offers an external multiparameter quality control (IMMUNO-TROL I, Cat: HM036; IMMUNO-TROL II, Cat: HM037). These controls should be handled as samples.

CHARACTERISTICS AND PERFORMANCES OF THE TEST

Sensitivity, specific affinity and agreement to other methods

ELISIS CARDIOLIPIN IgG/IgM kit (bmd) has been compared to one marketed ELISA kit.

The study was performed on 172 samples of:

⇒ 28 Systemic Lupus Erythematosus (SLE)

⇒ 36 Rheumatoid arthritis

⇒ 19 primary APS

⇒ 15 secondary APS

⇒ 3 connective tissue disease

⇒ 3 polyneuropathy

⇒ 1 cerebral vasculitis, 1 p-ANCA, 1 Sjögren syndrome, 1 prolonged reversible ischemic neurological deficiency, 1 fibromyalgia, 1 polyangiitis et 1 hypertension.

Results

		CARDIOLIPINE IgG Marketed kit	
		+	-
ELISIS CARDIOLIPIN IgG	+	42	1
	-	4	64

Relative sensibility : 91.3%
Relative specificity : 98.5%
Accuracy : 95.5%

		CARDIOLIPINE IgM		
		Marketed kit		
		+	-	
ELISIS CARDIOLIPIN IgM	+	18	1	Relative sensibility : 45% Relative specificity : 98.6% Accuracy : 79.3%
	-	22	70	

Discordant results

		Clinical diagnostic
IgG		- 3 patients with primary APS - 1 patient with other disease - 1 patient with secondary APS
IgM		- 5 patients with rheumatoid arthritis - 5 patients with SLE - 6 patients with primary APS - 3 patients with secondary APS - 4 patients with other disease

Among the IgM discordant samples, 9 patients present a clinical diagnosis (rheumatoid arthritis and other disease) which lets suppose that the concurrant marketed kit used, in this evaluation, present false positives.

Test precision

Estimated for IgG and IgM isotype on three positive samples : strongly, moderate and weakly positive.

ELISIS CARDIOLIPIN IgG		ELISIS CARDIOLIPIN IgM	
Intra-assay (24 tests in a same assay)		Intra-assay (24 tests in a same assay)	
Titer mean (GPL/mL)	CV (%)	Titer mean (MPL/mL)	CV (%)
586.2	1.5	499.8	0.9
67.4	3.4	68.9	1.7
34.5	7.6	40.7	4.6

ELISIS CARDIOLIPIN IgG		ELISIS CARDIOLIPIN IgM	
Inter-assays (18 tests in 3 differents assays)		Inter-assays (18 tests in 3 differents assays)	
Titer mean (GPL/mL)	CV (%)	Titer mean (MPL/mL)	CV (%)
156.2	0.4	110.2	0.7
37.0	0.8	24.0	0.7
11.4	2.6	11.8	2.7

Threshold value

Calculated on 40 samples from « healthy subjects ».

	ELISIS CARDIOLIPIN IgG/IgM
Cut-off : Mean value +3SD	15 GPL/mL Or 15 MPL/mL

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BMD Editions (1999).

SUMMARY OF METHOD

	AMOUNT TO BE DISTRIBUTED	REAGENTS		INCUBATION CONDITIONS
		QUANTITATIVE METHOD	QUALITATIVE METHOD	
INCUBATION OF SAMPLES	100µL	Calibrators A, B, C, D, E and F (in duplicate)	Negative Control (in duplicate)	30 min at RT
	100µL	Positive Control (in duplicate)	Cut-off calibrator (in duplicate)	
	100µL	Negative Control (in duplicate)	Positive Control (in duplicate)	
	100µL	Diluted samples (in duplicate)	Diluted samples (in duplicate)	
WASHING	Wash 3 times with washing buffer (300µL/ well)			
INCUBATION OF CONJUGATE	100µL	Anti-human IgG-HRP conjugate OR Anti-human IgM-HRP conjugate		30 min at RT
WASHING	Wash 3 times with washing buffer (300 µL/ well)			
ENZYME REACTION	100µL	TMB Substrate		30 min at RT protected from intense light
STOP REACTION	100µL	Stop solution (1M HCl)		5 min

RT: room temperature (+20°C/+32°C)

SYMBOLS USED

	Biological risks		Temperature limitation		Catalogue Number
	Consult instructions for use		In Vitro Diagnostic Medical Device		Batch Code
	Number of tests		Use by		EC Declaration of Conformity
	Sample Diluent		Stop Solution		Cut-off calibrator
	Wash Buffer		Calibrator		Microplate
	Control		Substrate		Conjugate

BioMédical Diagnostics SA

Office
Actipole 25
4 bld de Beaubourg
77435 Marne la Vallée Cx2
France

Tel : 33 1 64 62 10 12
Fax : 33 1 64 62 09 66

E-mail : support@bmd-net.com
Internet : www.bmd-net.com