ELISIS VASCULITIS Profile IgG

REF LIS 1301  12 x 8 tests

DÉFINITION

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BIBLIOGRAPHIE

HAGEN EC et al.

Development and standardization of solid-phase assays for the detection of anti-neutrophil cytoplasmic antibodies (ANCA) for clinical application: report of a large clinical evaluation study. Proceedings of the 6th International ANCA Workshop. Paris, 28 June - 1 July 1995.

Clin Exp Immunol, 101/1: 29, 1995.

JENNETTE JC et al.

Pathogenic potential of anti-neutrophil cytoplasmic auto-antibodies.

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RASMUSSEN N. et al.

An ELISA for the detection of antineutrophil cytoplasm antibodies (ANCA).

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Spectra Biologie, 93/3: 38-43, 1993.

SAXENA R et al.

Circulating autoantibodies as serological markers in the differential diagnosis of pulmonary renal syndrome.

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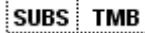
REF LIS 1301

DEFINITION

ELISIS VASCULITIS Profile IgG kit ([bmd](#)) is an enzyme linked immunoassay (ELISA) for the qualitative and semi-quantitative determination of IgG anti-proteinase 3 (PR3), anti-myeloperoxidase (MPO), anti-cathepsin, anti-elastase, anti-lactoferrin, anti-lysozym and anti-BPI (bacterial permeability-increasing protein) antibodies in human serum samples.

The results of the **ELISIS VASCULITIS Profile IgG kit** ([bmd](#)) are to be used in conjunction with the clinical findings and the other laboratory tests to aid in the differential diagnosis of autoimmune vasculitis.

ELISIS VASCULITIS Profile IgG kit ([bmd](#)) may be used with the **CARIS™ system** (diluting and dispensing device).

DIAGNOSTIC VALUE

The acronym ANCA (Anti-neutrophil cytoplasmic autoantibodies) describes a group of antibodies directed against different components of neutrophilic granulocytes and monocytes. For the detection of ANCAs indirect immunofluorescence test on ethanol-fixed neutrophils has been the established method so far. It became apparent that some ANCAs create a cytoplasmic fluorescence pattern (thus called cANCA) while others create a perinuclear pattern (the pANCA). As both patterns may cover multiple antigens, immunofluorescence does not suffice for a satisfying differential diagnosis of vasculitis; thus each IFT should be verified with specific ELISA tests.

Myeloperoxidase (MPO) has been identified as the major **pANCA** antigen, but other cellular components like **Lactoferrin, Cathepsin G, Lysozyme and Elastase** cause perinuclear staining, too and therefore are included into the group of pANCAs. **Proteinase 3** is the major target antigen of cANCA.

Detection of ANCAs is a useful laboratory diagnostic test for certain small vessel vasculitides and some non-vasculitic clinical syndromes, such as inflammatory bowel disease (IBD).

Antibodies against MPO correlate with idiopathic or vasculitis associated necrotizing crescentic glomerulonephritis. They are found frequently in 70% of patients with microscopic polyangiitis as well as 5-50% of patients with Churg-Strauss syndrome.

Antibodies against Lactoferrin and Cathepsin G were identified in a sub-group of patients with inflammatory bowel disease. However, these ANCA specificities do not appear to correlate with disease activity.

Autoantibodies against Elastase are generally associated with inflammatory rheumatic diseases like SLE, Sjögren's syndrome and Felty syndrome.

Antibodies against BPI are detected in chronically infectious intestinal diseases such as Morbus Crohn or ulcerative colitis. In contrast to anti-MPO anti-BPI antibodies do not seem to have any association with vasculitis. Antibodies to Lysozyme occur in higher frequency in rheumatoid vasculitis and inflammatory bowel disease like ulcerative colitis.

Autoantibodies to PR3 are a specific serological marker for Wegener's granulomatosis (WG).

ASSAY PRINCIPLE

Each antigen is individually coated onto the polystyrene microtiter plate (12 strips of 8 wells).

- First, the diluted sample is added to the coated well, which allows to bind. After incubation, unbound proteins are removed by washing.
- Then, a monoclonal anti-human IgG conjugated to horseradish peroxidase is added to each well. The conjugate binds to the antigen-antibody complex. After incubation, the excess conjugate is removed by a second wash.
- The chromogenic step is performed by the addition of TMB substrate (3,3',5,5' – tetramethylbenzidine). During this step, a colour will develop in proportion to the amount of antibodies in the sample.
- Addition of Stop Solution HCl (1 M) serves to stop the enzymatic reaction.
- After stopping the reaction by HCl (1 M) the optical density is read by a spectrophotometer at 450nm
- The rate of color formation from the chromogen is a function of the amount of conjugate bound to the antigen-antibody complex and this is proportional to the initial concentration of the respective antibodies in the patient sample.

SAMPLES COLLECTION AND HANDLING

- The test should be performed on serum. Use preferentially freshly collected serum samples. Blood withdrawal must follow national requirements.
- Lipemic sera should be avoided, as well as samples which have been frozen and defrosted more than once.
- If determination is not performed immediately, samples should be stored at +2°C/+8°C for no longer than a week or frozen.
- To avoid any non-specific fixation, samples which have been frozen for more than 6 months or which are cloudy, should be centrifuged and filtered.

STABILITY AND STORAGE

- Store reagents and assay strips at +2°C/+8°C in their own package.
- Do not use kits beyond the expiration date.
- Store unused strips into their plastic bag with the desiccant.
- Store all components immediately after use again at +2°C/+8°C.

REAGENTS

96 wells microplate individually coated with anti-proteinase 3 (PR3), anti-myeloperoxidase (MPO), anti-cathepsin, anti-elastase, anti-lactoferrin, anti-lysozym and anti-BPI antibodies (12x8 individual breakaway microwells)	MP	12 strips
1 vial of Cut-off Calibrator (capped blue: yellow solution) containing: human serum (diluted), sodium azide < 0.1% (preservative) Ready to use	CONTROL REF	1 x 1.5mL
4 vials of Calibrators A-D (color increasing with concentration: yellow solution) 0, 10, 30, 100 U/mL containing: human serum (diluted), sodium azide < 0.1% (preservative) Ready to use	CAL	4 x 1.5mL
1 vial of anti-human IgG-HRP Conjugate (capped blue: blue solution) Ready to use	CONJ IgG	1 x 15mL
1 vial of TMB Substrate (capped black) containing: stabilized TMB/ H ₂ O ₂ Ready to use	SUBS TMB	1 x 15mL
1 vial of Sample Diluent - 5x concentrated (capped white: yellow solution) containing: Tris, NaCl, BSA, sodium azide < 0.1% (preservative) To dilute	DIL SPE 5x	1 x 20mL
1 vial of Wash Buffer - 50x concentrated (capped white: green solution) containing: Tris, NaCl, Tween 20, sodium azide < 0.1% (preservative) To dilute	BUF WASH 50x	1 x 20mL
1 vial of Stop Solution (capped white: colorless solution) containing: 1M HydrochloricAcid Ready to use	SOLN STOP	1 x 15mL

ELISIS VASCULITIS Profile IgG kit ($\frac{\text{bmd}}{\text{bmd}}$) is calibrated in arbitrary units (AU/mL).

MATERIAL REQUIRED BUT NOT PROVIDED

- Purified water (according to the definition of the United States Pharmacopeia and the European Pharmacopeia)
- Microtiter plate reader 450nm reading filter and optional 620nm reference filter (600-690nm)
- Glass ware (cylinder 100-1000mL)
- Test tubes for serum dilutions
- Vortex mixer
- Precision pipettes (10, 100, 200, 500, 1000µL) or adjustable multipipette (100-1000µL)
- Microplate washing device (300µL repeating or multichannel pipette or automated System)
- Adsorbent paper

SETUP

All the reagents should be prepared as required:

1. Preparation of Sample Diluent Solution

- Dilute concentrated Sample Diluent 1/5 in distilled water.
Ex: 20mL plus 80mL
- Storage period: 1 month at +2°C/+8°C.

2. Preparation of wash buffer

- Dilute concentrated Wash buffer 1/50 in distilled water.
Ex: 20mL plus 980mL
- Prepare 20mL of diluted wash buffer for 8 wells or 200mL for 96 wells.
- Storage period: 1 month at +2°C/+8°C.

3. Preparation of samples

- Dilute to 1/101 in Sample Diluent (1x)
Ex: 1000µL Sample Diluent (1x) + 10µL serum
- Shake vigorously at the vortex.

4. Washing Step

- ❖ Automated washing:
 - Consider excess volumes required for setting up the instrument and dead volume of robot pipette.
- ❖ Manual washing:
 - Discard liquid from wells by inverting the plate.
 - Knock the microwell frame with wells downside vigorously on clean adsorbent paper.
 - Pipette 300µl of diluted wash buffer into each well, wait for 20 seconds.
 - Repeat the whole procedure twice again.

5. Microplates

- Calculate the number of wells required for the test.
- Remove unused wells from the frame, replace and store in the provided plastic bag, together with desiccant, seal tightly (+2°C/+8°C).

PRECAUTIONS

Allow all reagents and samples to come to room temperature (+18°C / +25°C) before handling. Check that all plates are well drained after each wash. Do not smoke, eat or drink when manipulating the kit. Do not pipette by mouth.

Human sources for the preparation of calibrators have been tested and found negative for antibody to HIV 1 and 2, antibody to hepatitis C virus and hepatitis B virus antigen. Nevertheless, no test can offer complete assurance that HIV, hepatitis B virus or other infectious agents are absent. Therefore, the reagents should be handled as potentially infective materials.

This kit contains potentially hazardous components. Though kit reagents are not classified being irritant to eyes and skin we recommend to avoid contact with eyes and skin and wear disposable gloves.

Calibrators and Buffers contain less of 0.1% (w/v) sodium azide. Do not eat and avoid contact with skin and eyes. This substance can form explosive mixtures in copper or lead piping. Rinse thoroughly after flushing.

ELISIS VASCULITIS Profile IgG kit ([bmd](#)) has been developed according CE Directives 67/548/EEC and 1999/45/EC relating to the classification, packaging and labeling of dangerous preparations.

ELISIS VASCULITIS Profile IgG kit ([bmd](#)) has been optimized for the use as describe in this procedure. Do not substitute other manufacturer's reagents. Dilution or adulteration of these reagents may also affect the performance of the test. Close adherence to the test procedure will assure optimal performance.

Do not mix or substitute reagents or microplates from different lot numbers. This may lead to variations in the results.

Incubation: We recommend test performance at +30°C for automated Systems.

Never expose components to higher temperature than +37°C.

Always pipette Substrate and Conjugate swith brand new tips only.

Protect Substrate from light.

METHOD

1. Preparing the test

Use the work sheet to note the sample locations.

Detach the exact number of wells needed and return unused wells to plastic pouch provided in the kit, with the desiccant bag.

For quantitative interpretation use calibrators to establish a standard curve

Set out:

- Calibrators (Cal A, Cal B, Cal C and Cal D)
- Diluted samples (S)

	1	2	3	4	5	6	7	8	9	10	11	12	
Calibrator	A	Cal.A	Cal.B	Cal.C	Cal.D								
PR3	B	S1	S2	S3	S4	S5	S6	S7	S8	S9	S10	S11	S12
MPO	C	S1	S2	S3	S4	S5	S6	S7	S8	S9	S10	S11	S12
BPI	D	S1	S2	S3	S4	S5	S6	S7	S8	S9	S10	S11	S12
Elastase	E	S1	S2	S3	S4	S5	S6	S7	S8	S9	S10	S11	S12
Cathepsin G	F	S1	S2	S3	S4	S5	S6	S7	S8	S9	S10	S11	S12
Lysozym	G	S1	S2	S3	S4	S5	S6	S7	S8	S9	S10	S11	S12
Lactoferrin	H	S1	S2	S3	S4	S5	S6	S7	S8	S9	S10	S11	S12

For qualitative interpretation use cut-off calibrators

Set out:

- Calibrator A (Ctrl -)
- Cut-off calibrator (Ctrl Ref)
- Calibrator D (Ctrl +)
- Diluted samples (S)

	1	2	3	4	5	6	7	8	9	10	11	12	
Calibrator	A	Cal.A	Ctrl Ref	Cal.D									
PR3	B	S1	S2	S3	S4	S5	S6	S7	S8	S9	S10	S11	S12
MPO	C	S1	S2	S3	S4	S5	S6	S7	S8	S9	S10	S11	S12
BPI	D	S1	S2	S3	S4	S5	S6	S7	S8	S9	S10	S11	S12
Elastase	E	S1	S2	S3	S4	S5	S6	S7	S8	S9	S10	S11	S12
Cathepsin G	F	S1	S2	S3	S4	S5	S6	S7	S8	S9	S10	S11	S12
Lysozym	G	S1	S2	S3	S4	S5	S6	S7	S8	S9	S10	S11	S12
Lactoferrin	H	S1	S2	S3	S4	S5	S6	S7	S8	S9	S10	S11	S12

2. Sample incubation

Add 100µL of Calibrators OR Cut-off Calibrator and controls into the designated microwells

Add 100µL of diluted samples

Incubate for 30 minutes at room temperature (+20°C/+32°C)

Wash step:

Remove the contents of the wells by rapid inversion

Wash 3 times with 300µL of washing buffer

3. Incubation of conjugate

Add 100µL of conjugate

Incubate for 30 minutes at room temperature (+20°C/+32°C)

Wash step:

Remove the contents of the wells by rapid inversion

Wash 3 times with 300µL of washing buffer

4. Incubation of substrate

Add 100µL TMB substrate into each well

Incubate for 30 minutes at room temperature (+20°C/+32°C)

Protect the wells from intense light

5. Stop solution

Add 100µL of Stop solution to each well

Incubate for 5 minutes minimum

Agitate plate carefully for 5 sec.

6. Reading

Read the optical density of each well at 450nm (optionally 450/620nm) within 30 minutes.

INTERPRETATION OF RESULTS

For **quantitative interpretation** establish the standard curve by plotting the **optical density (OD) of each calibrator (y-axis)** with respect to the corresponding concentration values in **AU/mL (x-axis)**. For best results [bmd](#) recommend log (x) / lin (y) coordinates and 4-Parameter Fit. From the OD of each sample, read the corresponding antibody concentrations expressed in **AU/mL**.

Negative Results	Positive Results
≤ 15 AU/mL	> 15 AU/mL

Example of a standard curve

We recommend pipetting calibrators in parallel for each run.

Calibrators IgG	OD 450/620nm
0 AU/ml	0.040
10 AU/ml	0.350
30 AU/ml	0.680
100 AU/ml	1.220

Do not use this example for interpreting patient's results!

For lot specific data, see enclosed quality control certificate.

Each laboratory should establish its own normal range based upon its own techniques, controls, equipment and patient population according to their own established procedures.

For **qualitative interpretation** read the optical density of the cut-off calibrator and the patient samples. Compare patient's OD with the OD of the cut-off calibrator. All samples which are higher than cut-off are considered positive.

Negative:	OD Patient < OD cut-off
Positive:	OD Patient > OD cut-off

A definite clinical diagnosis should not be based on the results of the performed test only, but should be made by the physician after all clinical and laboratory findings have been evaluated. The diagnosis is to be verified using different diagnostic methods.

LIMITATION

Hemolytic, lipemic, icteric samples or contaminated samples with may confound the results of this assay.

QUALITY CONTROL

[bmd](#) offers an external multiparameter quality control (IMMUNO-TROL I, Cat: HM036). This control should be handled as samples.

REFERENCES

HAGEN EC et al.

Development and standardization of solid-phase assays for the detection of anti-neutrophil cytoplasmic antibodies (ANCA) for clinical application: report of a large clinical evaluation study. Proceedings of the 6th International ANCA Workshop. Paris, 28 June - 1 July 1995. *Clin Exp Immunol*, 101/1: 29, 1995.

JENNETTE JC et al.

Pathogenic potential of anti-neutrophil cytoplasmic auto-antibodies. *lab Invest* 70: 135-137, 1994.

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RASMUSSEN N. et al.

An ELISA for the detection of antineutrophil cytoplasm antibodies (ANCA). *J Immunol Meth* 127: 139-145, 1990.

SANCHEZ-LALLOYER N.

Les anticorps anti-cytoplasme des polynucléaires neutrophiles. *Spectra Biologie*, 93/3: 38-43, 1993.

SAXENA R et al.

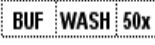
Circulating autoantibodies as serological markers in the differential diagnosis of pulmonary renal syndrome. *J.Intern. Med.* 1995, 237, 143-152

SUMMARY OF METHOD

	AMOUNT TO BE DISTRIBUTED	REAGENTS		INCUBATION CONDITIONS
		QUANTITATIVE METHOD	QUALITATIVE METHOD	
INCUBATION OF SAMPLES	100µL	Calibrator A	Calibrator A	30 min at RT
	100µL	Calibrator B	Cut-off calibrator	
	100µL	Calibrator C	Calibrator D	
	100µL	Calibrator D	Diluted samples	
	100µL	Diluted samples		
WASHING	Wash 3 times with washing buffer (300µL/ well)			
INCUBATION OF CONJUGATE	100µL	Anti-human IgG-HRP conjugate		30 min at RT
WASHING	Wash 3 times with washing buffer (300µL/ well)			
ENZYME REACTION	100µL	TMB Substrate		30 min at RT protected from intense light
STOP REACTION	100µL	Stop solution (1M HCl)		5 min

RT: room temperature (+20°C/+32°C)

SYMBOLS USED

	Biological risk		Temperature limitation		Catalog Number
	Read instructions for use		In Vitro Diagnostic Use		Lot Number
	Number of tests		Use by		EC Declaration of Conformity
	Sample Diluent		Stop Solution		Cut-off calibrator
	Wash Buffer		Calibrator		Microplate
	Control		Substrate		Conjugate

BioMédical Diagnostics SA

Office
Actipole 25
4 bld de Beaubourg
77435 Marne la Vallée Cx2
France

Tel : 33 1 64 62 10 12
Fax : 33 1 64 62 09 66

E-mail : support@bmd-net.com
Internet : www.bmd-net.com