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REF | ME 0131 Substrat slide (4 wells)


INTENDED USE

The section of rat esophagus is used for the screening anti-keratin antibodies in Rheumatoid Arthritis (RA) patients using indirect immunofluorescence method.

DIAGNOSTIC VALUE

The polyarthritis rheumatoid is most frequently observed chronic inflammatory rheumatism. The anti-keratin antibodies are a specific marker of the polyarthritis rheumatoid. They are present in 50% of seropositive polyarthritis and 20% of the seronegatives forms, but are not found in other inflammatory rheumatism.

These antibodies are bind on the keratinized stratified squamous epithelium which surround the light of the lower third of the esophagus.

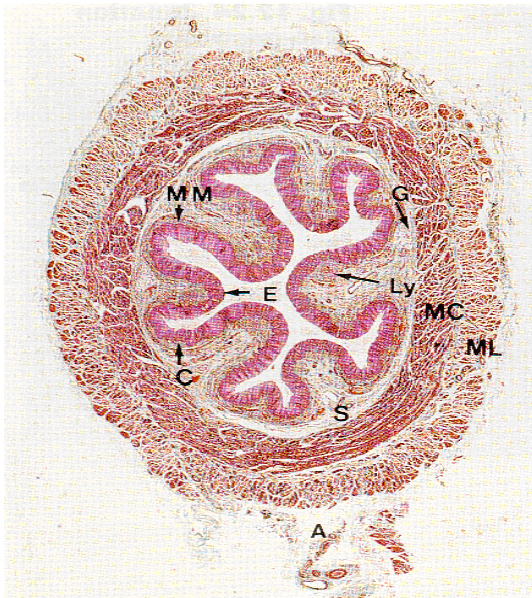This section is doing in lower third of the esophagus and shows the basic layers of the gastro-intestinal tract. The lumen is surrounded by the tunica mucosa (E); the lining of the esophagus in all species is stratified squamous epithelium, and in most animals (like rodents) it's keratinized, sometimes quite heavily. The underlying chorion C contains lymphoid cells (Ly). Although few details are visible at this very low magnification, you can see that it is a muscular tube lined with epithelium (MM). The submucosa (S) secretes mucus from mucous glands (G) which are observed primarily only on the lower third area of the oesophagus. The tunica muscularis is clearly defined as two layers, one inner (MC) and one outer (ML) composed of smooth muscle fibers that separates the tunica mucosa from the tunica submucosa. Because the first part of swallowing is under voluntary control, the beams of stratified muscle prevail in the muscularis one of the higher third of the esophagus. Outboard of tunica muscularis, the "tunica adventitia" A, very richly vascularized is continuous by the adjacent areolar conjunctive tissue. Esophageal mucous membrane relaxed presents many deep folds; this layout allows an important distension during the passage of food.

PRINCIPLE OF THE TEST

Anti-keratin antibodies are detected by indirect immunofluorescence on sections of rat oesophagus (made in the keratinized lower third) with a fluorescent conjugate.

First:

Sections of rat esophagus on microscope slides are overlaid with diluted patient's serum and incubated to allow the anti-keratin antibodies – present in positive sera – to bind to the antigen sites in the oesophagus tissue.

Second:

Sites where the anti-keratin antibodies reacted with the substrate are made visible by the binding of the fluorescently labelled anti-human IgG globulin (the conjugate).

COMPONENTS

		ME 0131
Substrate Slide (section of rat esophagus in multiple well-slides)	SORB SLD 4	1 slide of 4 wells
Blotter for drying the substrate slides around the wells.		1

REAGENTS AVAILABLE INDIVIDUALLY

ME 902.2	Fluorescent anti-IgG conjugate (2 mL)
ME 902.8	Fluorescent anti-IgG conjugate (8 mL)
IMM 2245	Anti-keratin positive control (0,5mL)
HME NEG	Negative control (500 µL)
HME MM	Mounting Medium buffered glycerol (3,5mL)
HME PBS10	One-liter packet of dry PBS 10g (10x1L)
HME 12CS	24x60mm glass coverslips (100 units)

MATERIAL REQUIRED BUT NOT PROVIDED

- Volumetric pipettes to deliver 10-15 µL
- Small test tubes and rack or U-bottom microtitration plates
- Humid chambers – Petri dishes with moist filter paper
- Coplin staining jars with slide holders
- One liter volumetric flask or graduated cylinder
- Squeeze bottle for PBS
- Cover glasses #1, 24 x 60 mm.
- Distilled water
- Fluorescence microscope
- Timer
- Forceps

SPECIMEN COLLECTION

- Collect blood specimens in plain untreated tubes.
- After clot formation, immediately harvest the serum.
- The serum specimens may be stored in a refrigerator (+2°C-+8°C) for up to 48 hours. If longer storage is desired, store at -20°C.
- Turbidity, haemolysis, bacterial growth or drugs capable of fluorescing in the serum may adversely affect the performance of the test. Do not use hemolyzed or lipemic sera.
- Specimen should be handled as though they contained agents infectious to humans

STABILITY AND STORAGE

- Expiration dates for substrates (slides), conjugate and controls are given for storage at -20°C or below.

We recommend to use with this substrate, the reagents listed above § "REAGENT AVAILABLE INDIVIDUALLY".

- Once thawed the conjugate is stable for 90 days when stored at +2° to +8°C. Do not alicote the conjugate at -20°C.
- Repeated freezing and thawing should be avoided since it is detrimental to the products.
- The mounting medium is stable until the expiration date when stored in the refrigerator.
- The PBS powder can be stored at room temperature until its expiration date.

PRECAUTIONS

All bmd Test Kit components are for *in vitro* diagnostic use ONLY.

Do not use components or reagents beyond their expiration dates. All reagents must be at room temperature before use (at least 15 mn).

After freezing, use each slide immediately after opening the envelope to avoid the drying of the wells.

In order to avoid damaging the substrate:

- Apply each drop on the surround of wells in order to touch the substrate with the tip of the pipette.
- During the wash step, do not rince the slides with a pipette or squeeze (wash) bottle.
- Do not apply any pressure on the slide during to mount the cover glasses.
- Do not move the substrate slide after adding the mounting medium buffered.
- Avoid using pen to identify the slide.
- **Always avoid mouth pipetting.**

SET-UP

All substrate slides must be at room temperature before use.

1. Preparation of the wash and dilution buffer

- Reconstitute buffer pack with one liter distilled water. Wait 10 mn before use
- Storage period : 4 weeks at +2°C/+8°C

2. Sera dilution

- Dilute freshly collected and properly stored patient serum 1:10 in PBS.
For titrations, set up doubling dilutions, i.e. 1:20, 1:40, etc.
- Mix with vortex

TEST PROCEDURE

1. Incubation of samples

- Identify wells which are going to contain positive and negative control sera and patient's serum dilutions.
- Remove slide from protective envelopes and put into the humid chamber
- In the identified well of the slide, add
 - **40µL** of diluted sample, **by avoiding to touch the substrate with the tip of the pipette.**
 - 1 drop of ready to use positive control
 - 1 drop of ready to use negative control
- Do the same with each slide.
- Incubate for 30 minutes at room temperature.

Do not let the wells dry out

2. Wash with PBS

- Remove slides from humid chamber.
- Tap each slide quickly on its side to avoid contaminations.
- Dip the slides several times in a beaker containing fresh PBS. **Do not use wash bottle to rinse the slides.**
- Immediately place slides in a slide holder or a coplin jar and wash with PBS and incubate for 10 minutes at room temperature. Agitation is advised but not useful.

Note: If more than 10 slides are tested, wash two time in PBS bath (5 minutes each).

⚠ Extended incubation and improper washing may result in poor morphology of substrate and increased background fluorescence.

3. Incubation of Conjugate

- Remove slides from PBS.
- Eliminate excess of PBS onto some absorbent paper.
- Blot the slides with blotters provided to dry around the wells and put into the humid chamber.
- Immediately apply 1 drop of conjugate to each well to cover substrates.
- Incubate for 30 minutes at room temperature and **far from light.**

Do not let the wells dry out

4. PBS rinse

- Repeat Step 2 "Wash with PBS".

5. Mount Cover glasses

- Apply 3 or 4 drops mounting medium to each slide.
- Cover with cover glasses – Lateral movement and pressure will damage the substrate

6. Reading

It is recommended to examine the day of the test with an appropriately equipped fluorescence microscope (x10) and use microscopes (x25 or x40) to confirm a positive fluorescence.

Note: If delay in examining slides is anticipated, seal coverglass with clear nail polish and store in refrigerator.

INTERPRETATION OF RESULTS

Starts the reading with a dilution of 1:10. For all positive readings at dilution of 1:10 : doubles with every successive dilution, so the next specimen tested is 1:20, then 1:40, 1:80, and 1:160.

Auto-antibodies detected	Cutt off
Anti-Keratin antibodies	$\geq 1/10$

Positive reaction

A positive reaction results in apple green fluorescence pattern in the stratum corneum of rat esophagus epithelium.

All positive results should be tested again 2 months later and interpreted according to the biological and clinical context.

Negative reaction

A simple apple/green fluorescence border should not be interpreted as positive.

QUALITY CONTROL

When results obtained are distant from expected values, test run should be repeated. Should atypical observations continue, we recommend to investigate if :

- adequate amount of serum is applied on the wells
- wells were allowed to dry out during incubation of serum or conjugate
- there is any sign of microbial contamination of the liquid reagents due to incorrect storage and handling. If turbidity appears on any of the liquid reagents, discard and use another batch.
- Fluorescence microscope is properly aligned, using correct bulb, correct filters, etc...

bmd anti-keratin positive control (Cat. IMM 2245) can be used as an external quality control.

LIMITATIONS OF THE TEST

The **substrate slide of rat esophagus** is a valuable diagnostic aid. However, it should not be considered diagnostic by itself alone. All tests should be interpreted by a medical authority in light of all patients conditions.

Sensitivity of the test kit may be influenced by several factors aside from the components. We recommend user to take note of factors such as optical system, improper storage and handling, technician / observer, etc.

Samples that were screened positive at 1:10 should be further diluted. All positive sera should be titrated to the highest titre using a series of doubling dilutions, i.e. 1:20, 1:40, etc. The end point titre is the highest dilution that produces a positive result.

EXPECTED VALUES

Endomysial antibodies are present in almost 100% patients with celiac disease, 70% of patients with dermatitis herpetiformis, and in about 20% of patients with other enteropathies^(3,4).

There is a small percentage of IgG AEmA that will be negative IgA when screened. A negative result exhibited by a patient with overt clinical symptoms may be considered for IgG testing.

PERFORMANCE CHARACTERISTICS

The study was performed on 22 serums with a method of immunofluorescence indirect. The bmd substrate slide of rat esophagus (A) were tested in parallel with the substrate slide of rat esophagus of another supplier (B). Samples were diluted at 1/20.

Results of this evaluation are reported in the following table:

APPEARENCE	Slides (A)	Sildes (B)
<i>Positive</i>	7	7
<i>Negative</i>	15	15

This study was made on two different series and with two different batches, in order to test the reproducibility. Positive control appeared positive in each series with the substrate slide (A). Qualitative reproducibility is good.

Relative sensibility(7/7) : 100%

Relative specificity (14/14) : 100%

Concordance of results (22/22) : 100%

BIBLIOGRAPHY

1. Paimela L. ; et al. Antikeratin antibodies : diagnostic and pronostic markers for early rheumatoid arthritis. Annals of Rheumatic diseases 1992 ; 51 ; 743-746.
2. Youinou P., Le Roux A.M. Les anticorps anti-kératine réhabilités. Le concours médical, 12-10-1985-107-36pp 3406-3408.
3. Vincent C ; Youinou P ; Serre Guy (1999) – Auto-anticorps anti filaggrine – Autoanticorps marqueurs des maladies auto-immunes – bmd Editions Elsevier p 225.
4. Humbel RL (1994) – Connectivites – Auto-anticorps et maladies auto-immunes. Editions Option Bio Elsevier version N°1 – p.71.
5. Meyer O. (2003) – Devant un rhumatisme inflammatoire, quels auto-anticorps demander et quand ? Revue du rhumatisme 2003, vol 70, n°10-11 pp 803-817.

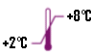
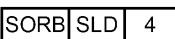
SUMMARY OF METHOD

	AMOUNT TO BE DISTRIBUTED	REAGENTS	INCUBATION CONDITIONS
INCUBATION OF SAMPLES	40 µL	Diluted samples	30 mn at R.T
	1 drop	Positive control (ready to use)	
	1 drop	Negative control (ready to use)	
<i>Do the same with each slide</i>			
PBS WASH	<ul style="list-style-type: none"> ⇒ Tap each slide quickly on its side or onto absorbent paper to avoid contaminations ⇒ Dip the slides several times in a beaker containing fresh PBS ⇒ Place slides in a slide holder or a coplin jar and wash with PBS. N.B : Agitation is not useful. 		10 mn at R.T
<i>Eliminate excess of PBS with the blotters provided (*) – Dry uniformly and without friction using a roller</i>			
INCUBATION OF CONJUGATE	1 drop to each well	Fluorescein conjugated Affinipure Goat anti-human IgG (ready to use)	30 mn at R.T in the dark
<i>Verify that all wells are perfectly covered</i>			
PBS RINSE	<ul style="list-style-type: none"> ⇒ Tap quickly each slide on its side to eliminate the conjugate. ⇒ Dip the slide several times in a beaker containing fresh PBS. ⇒ Place slides in a slide holder or a coplin jar and wash with PBS. N.B : Agitation is not useful. 		10 mn at R.T
<i>DO NOT ELIMINATE EXCESS OF PBS (Teflon) (*)</i>			
MOUNT COVER GLASSES	3-4 drops to each slide	Mounting Medium (ready to use)	-
READING	Examine with an appropriately equipped fluorescence microscope (x 10 – x 25 - x 40).		

(*) **Do not let wells dry out.** Every deposit must be made at once.

R.T : Room Temperature

SYMBOLS USED

	Biological risks		Temperature limitation		Catalogue Number
	Consult instructions for use		In Vitro Diagnostic Medical Device		Batch Code
	Number of tests		Use by		EC Declaration of Conformity
	Slide of 4 wells		Immunofluorescent assay		

BioMédical Diagnostics SA

Office
Actipole 25
4 bld de Beaubourg
77435 Marne la Vallée Cx2
France

Tel : 33 1 64 62 10 12
Fax : 33 1 64 62 09 66

E-mail : support@bmd-net.com
Internet : www.bmd-net.com